


## How to Assemble Your Face Shield

Our GCP.2 Face Shield arrives flat packed to reduce shipping costs and storage issues. The unit should be personally assembled by the end-user to avoid cross-contamination. Following the **eight simple steps**, the process takes around a minute to complete.

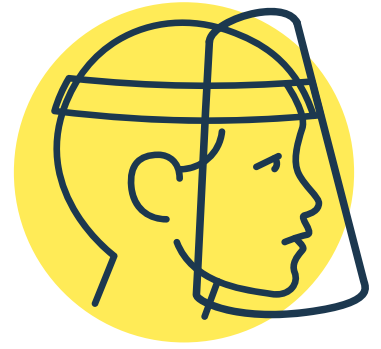

### STEP 1 OF 8

#### Unpack the Product



Open the sealed polythene bag containing:

- Perforated flat sheet PPE face shield
- Adjustable head strap
- 3x plastic rivets
- Soft water repellent foam strip with strong self-adhesive backing

### STEP 2 OF 8

#### Fold Along the Perforations



Carefully fold the visor along the perforated edges. Once along the straight perforation and once along the curved perforation of the forehead guard.

### STEP 3 OF 8

#### Position the Strap



Locate the adjustable strap, and identify the end with only one hole. Align this hole with one of the holes on the face shield.

### STEP 4 OF 8


#### Snap Together



Attach the strap to the product using one of the supplied plastic rivets. The rivet will go through the front of the shield first, through the strap hole and then again through the folded down forehead guard. The plastic rivet snaps together with hand pressure.

## Steps 5 – 8

---

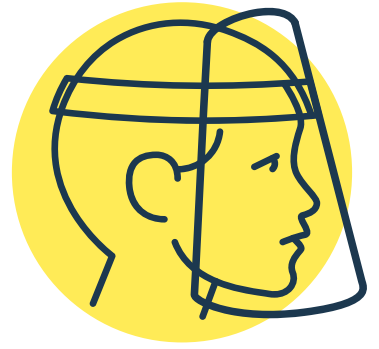

### STEP 5 OF 8

#### Set Forehead Shield and Curve

---



Using another plastic rivet, and the next set of 5 holes on the strap, set the head strap position on the opposite side of the face shield. The 5 holes allow you to adjust the curvature of the mask and forehead guard as desired.

### STEP 6 OF 8

#### Attach the Foam Strip

---



Remove the backing from the foam strip and position this along the forehead strap. Again, use hand pressure to ensure the foam has attached firmly.

### STEP 7 OF 8

#### Measure Up

---



Now place the assembled face shield around your head to measure up the final strap position on the remaining 7 holes.

### STEP 8 OF 8

#### Final Fit

---



Now snap the final plastic rivet into place to complete the assembly.